



2 Fold Stretcher



Golden Season's Folding Stretcher is widely used by Paramedics, EMS Rescue Professionals, Ambulance Services and other health care professionals. The lightweight stretcher is built from strong and high quality materials that allows it to withstand a high payload. It is tested by 3rd Party test centres to fail only at static load of above 800 kg. The mid-point deflection is only approximately 2 cm with a 120 kg static load.

Key Features at glance:

- Compact
- Rugged
- Lightweight
- Reliable
- Waterproof
- Easy to Clean

Specifications	
Dimension when open:	219cm x 53cm x 13.5cm
Dimension when folded:	109cm x 53cm x 17cm
Dimension when closed:	109cm x 9cm x 19cm
Weight:	5.97kg
Material Type:	Lightweight Aluminium
Accessories:	2 Patient restrain

*Specifications are subject to change without prior notice.

Golden Season Pte Ltd



2 Fold Stretcher - Setup Instructions

Step 1




- Check and ensure that you have a folding stretcher and a set of patient restraints.
- Remove the rubber Lock.

Step 2



- Unfold and extend the stretcher with the support leg facing up as shown above.

Step 3

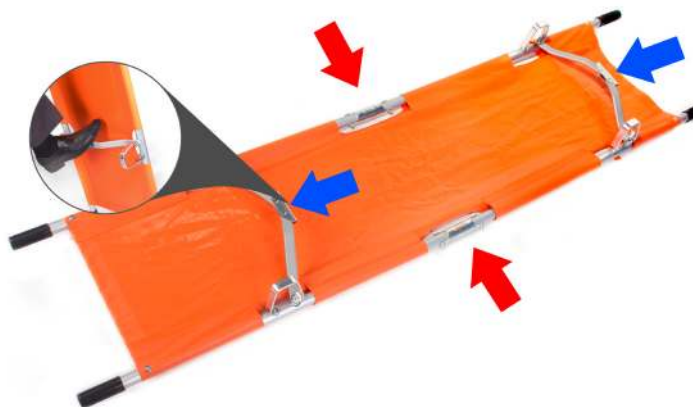

- Extend the stretcher sideways and lock both hinge of the cross-bar by kicking the hinge in the direction of the red arrow as shown the image above.
- Ensure the hinges are locked properly for safety usage.